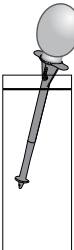
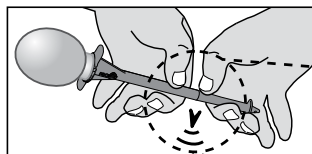
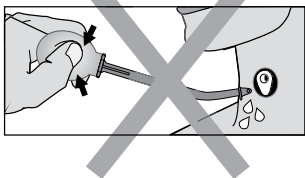
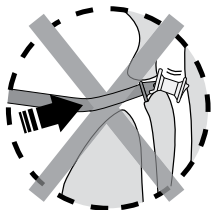
PROVOX[®] Flush

→


1	2	3	4	5	6	7
8	9	10	11	12	13	14
15	16	17	18	19	20	21
22	23	24	25	26	27	28
29	30					

x 12



→


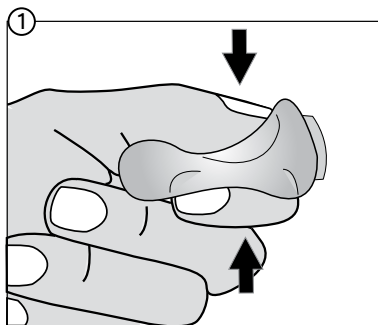
Max use 12 months


+

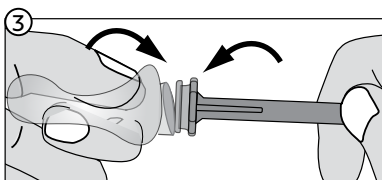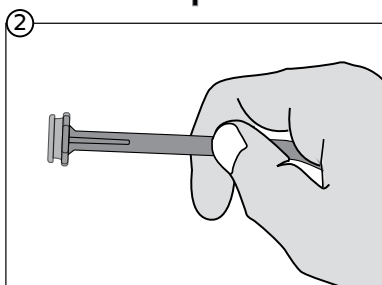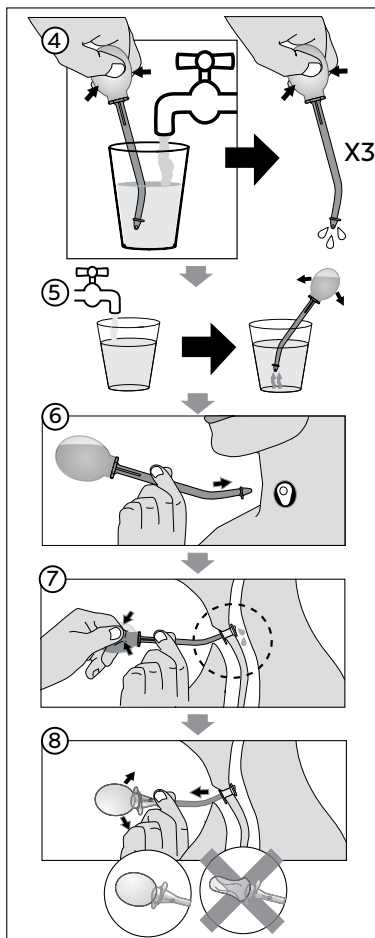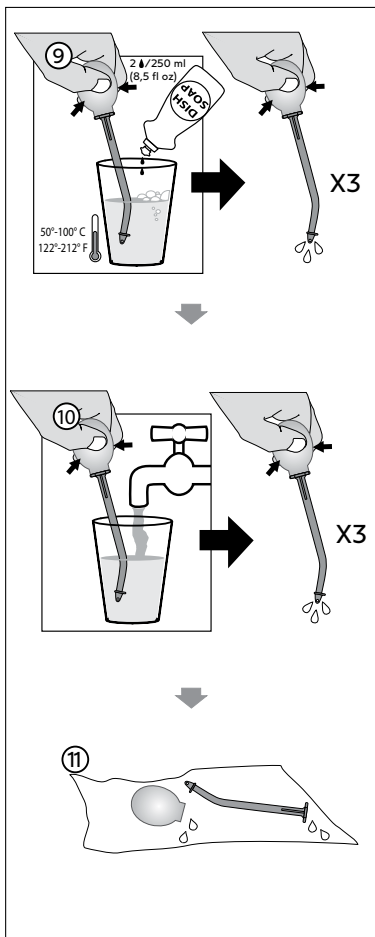12

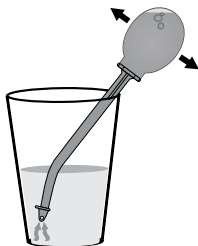
70 % Ethanol  10 min

or

70 % Isopropylalcohol  10 min

or

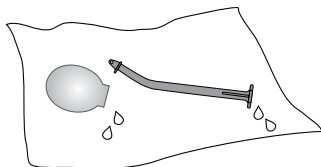3 % Hydrogenperoxide  60 min

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox® is a registered trademark owned by Atos Medical AB, Sweden. For information about protective rights (e.g. patents), please refer to our web page www.atosmedical.com/patents.

Manufacturer; Hersteller; Fabrikant; Fabricant; Produttore;
 Fabricante; Fabricante; Tillverkare; Producent; Produsent; Val-
 mistaja; Framleiðandi; Tootja; Ražotājs; Gamintojas; Výrobce;
 Gyártó; Výrobca; Proizvajalec; Producent; Producător;
 Proizvođač; Proizvođač; Κατασκευαστής; Производител; Üre-
 tici; მწარმოებელი; Αρτυσηρη; İstehsalçı; Производитель;
 Produsen; Pengilang; 製造業者; 제조사; 製造商; 制造商;
 היצר; الجهة المصنعة

Date of manufacture; Herstellungsdatum; Productiedatum; Date
 de fabrication; Data di fabbricazione; Fecha de fabricación;
 Data de fabrico; Tillverkningsdatum; Fremstillingsdato; Prod-
 uksjonsdato; Valmistuspäivä; Framleiðsludagsetning; Valmis-
 tamise kuupäev; Ražošanas datums; Pagaminimo data; Datum
 výroby; Gyártás időpontja; Dátum výroby; Datum proizvodnje;
 Data produkcji; Data fabricației; Datum proizvodnje; Datum
 proizvodnje; Ημερομηνία κατασκευής; Дата на производство;
 Üretim tarihi; წარმოების თარიღი; Артышһрнһн Һнһһһһһһ;
 İstehsal tarixi; Дата изготовления; Tanggal produksi; Tarikh
 pembuatan; 製造日; 제조일자; 製造日期; 制造日期;
 תאריך ייצור; تاريخ التصنيع

Use-by date; Verfallsdatum; Houdbaarheidsdatum; Date limite
 d'utilisation; Data di scadenza; Fecha de caducidad; Data de
 validade; Används senast: Användes inden; Brukes innen;
 Viimeinen käyttöpäivä; Sīdasti notkunardagur; Kõlblik kuni;
 Derīguma termiņš; Naudoti iki; Datum expirace; Felhasznál-
 hatóság dátuma; Dátum najneskoršej spotreby; Rok uporabnosti;
 Data ważności; Data expirării; Rok uporabe; Rok upotrebe;
 Ημερομηνία λήξης; Срок на годност; Son kullanım tarihi;

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Intended use

The Provox Flush is intended to be used to flush drinking water or air through the inner lumen of a Provox voice prosthesis for cleaning purposes. The Provox Flush is intended for both home and clinical use by patient or clinician.

WARNINGS

- The flush is only recommended for patients with sufficient manual dexterity, acceptable vision and satisfactory cognitive ability.
- Do not put pressure on the voice prosthesis, the voice prosthesis may be dislodged into esophagus.
- Do not squeeze the bladder before the flush is placed into the voice prosthesis. Water will enter the trachea leading to coughing. Let the bladder be refilled with air before removing the flush from the voice prosthesis (fig 8).

Cleaning and disinfection

Clean the Provox Flush after each use according to fig. 9-11.

Do not use any water other than drinking water to clean and rinse the device.

Disinfect the Provox Flush at least once a day (fig. 12) with one of the following methods:

- Ethanol 70% for 10 minutes
- Isopropylalcohol 70% for 10 minutes
- Hydrogenperoxide 3% for 60 minutes

Caution: Do not clean or disinfect by any other method as this might cause product damage and injury.

If the Provox Flush looks dirty or has air dried in an area with a risk of contamination the device should be both cleaned and disinfected before use. A risk of contamination could exist if the device has been dropped on the floor or if it has been in contact with a pet, someone with a respiratory infection, or any other gross contamination.

During hospitalization it is important to both clean and disinfect the Provox Flush after use but also **before** use since there is an increased risk of infection and contamination. In a hospital it is better to use sterile water for cleaning and rinsing, rather than drinking water.

Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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